



Compatibility table of the CholangoFlex™ Confocal Miniprobe™ with ERCP catheters

PRODUCT	REFERENCE	INTERNAL DIAM.	OPERATING PROCEDURE
Fusion® Oasis® Introduction System	Cook Medical <i>FS-OA-8.5</i>	8.5 Fr 2,83 mm	 <ul style="list-style-type: none"> • Proceed to ERCP as per standard of care • Once the catheter is in the bile/pancreatic duct at the desired location, remove the catheter from the duodenoscope, leaving the guidewire in place • Preload the Confocal Miniprobe in the catheter and re-insert the catheter and Confocal Miniprobe (along the guidewire) in the bile/pancreatic duct • Once at the desired location, push the Confocal Miniprobe out of the catheter and start imaging • After imaging, remove the Confocal Miniprobe (along with the catheter in case of resistance)
Fusion® Pushing Catheter	Cook Medical <i>FS-PC-5</i> <i>FS-PC-7</i>	5 Fr/1,66 mm 7Fr/2,33mm	
Cotton® Graduated Dilation Catheter*	Cook Medical <i>CGDC-8.5-7-5</i>	8.5 Fr 2,83 mm	
Huibregtse-Katon® ERCP Catheter	Cook Medical <i>ERCP-1-HKB</i>	5 Fr 1,88 mm	
Swing Tip Cannula	Olympus <i>PR-233Q</i>	4 Fr 1,33 mm	
Infinity™	US Endoscopy <i>711652</i>	9 Fr 3mm	
Web® II Double Lumen Extraction Basket	Cook Medical <i>MSB-35-2X4</i>	8 Fr 2,66 mm	 <ul style="list-style-type: none"> • Remove the accessory from the catheter • Preload the Confocal Miniprobe in the accessory channel of the catheter • Introduce the catheter into the duodenoscope, and proceed to ERCP as per standard of care • Once at the desired location, push the Confocal Miniprobe out of the catheter and start imaging • After imaging, remove the Confocal Miniprobe (along with the catheter in case of resistance)
Fusion® Cytology Brush	Cook Medical <i>FS-CB-1.5-S</i>	8 Fr 2,66 mm	
Howell Biliary Introducer	Cook Medical <i>HB-1</i>	10 Fr 3,33 mm	
Cytomax II Biliary Cytology Brush	Cook Medical <i>DLB-35-1.5-S</i>	6 Fr 2mm	

Compatibility table of the CholangioFlex™ Confocal Miniprobe™ with ERCP catheters

PRODUCT	REFERENCE	INTERNAL DIAM.	COMMENTS	
Soehendra® Biliary Dilation Catheter	Cook Medical <i>SBDC</i>	8.5 Fr 2,83 mm		<div style="border: 1px solid gray; border-radius: 10px; padding: 2px; display: inline-block;">1 lumen</div>  The catheter is too flexible, and does not protect the Confocal Miniprobe properly